# PANTALLA FACIAL FACE SHIELD

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- > EASY TO ASSEMBLE.
- > COMFORTABLE AND LIGHTWEIGHT.
- > EASY TO CLEAN.
- > REUSABLE.
- > STRONG.
- > CUSTOMIZABLE.
- > MADE IN EUROPE.

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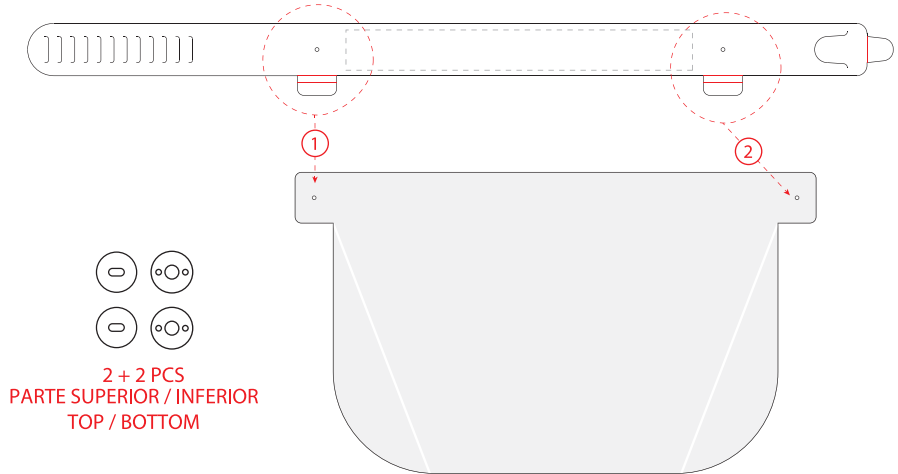
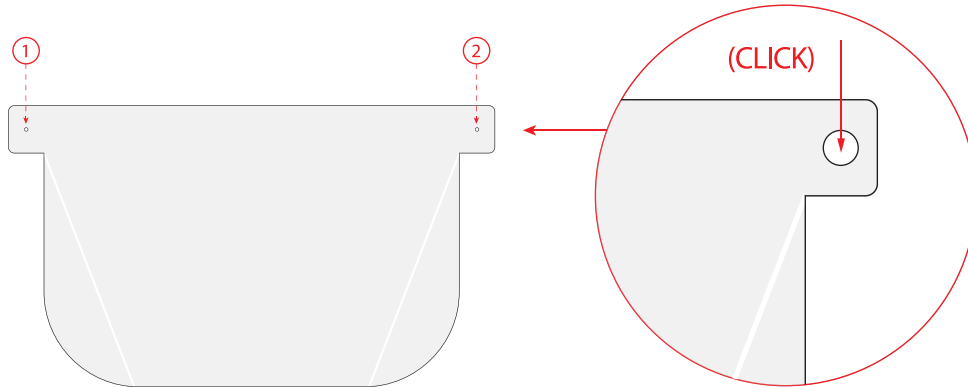
## ATTACHING THE TWO BUTTONS ONTO THE TRANSPARENT FACE SHIELD


Place the top part of the button over the holes of the face shield and press the stem through each of the holes.

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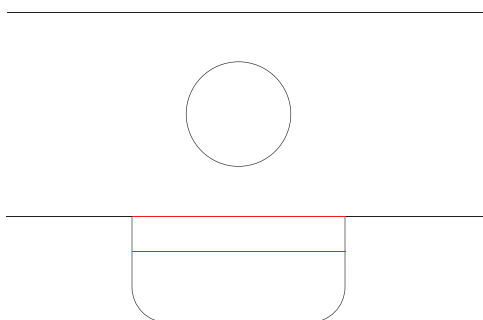
x 2 SOLAPAS DE LA ABRAZADERA  
x 2 TABS

## TO FASTEN THE FACE SHIELD

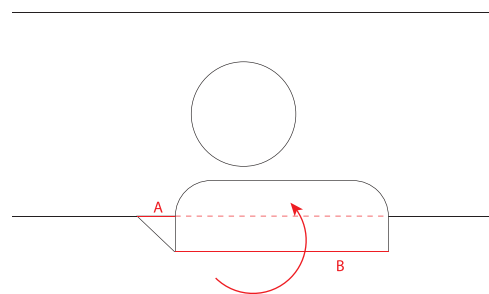Bend the two tabs on the head strap outwards forming a U shape. (Bend the opposite way to the black rubber strip).

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“ We recommend clean the shield frequently with alcohol. ”

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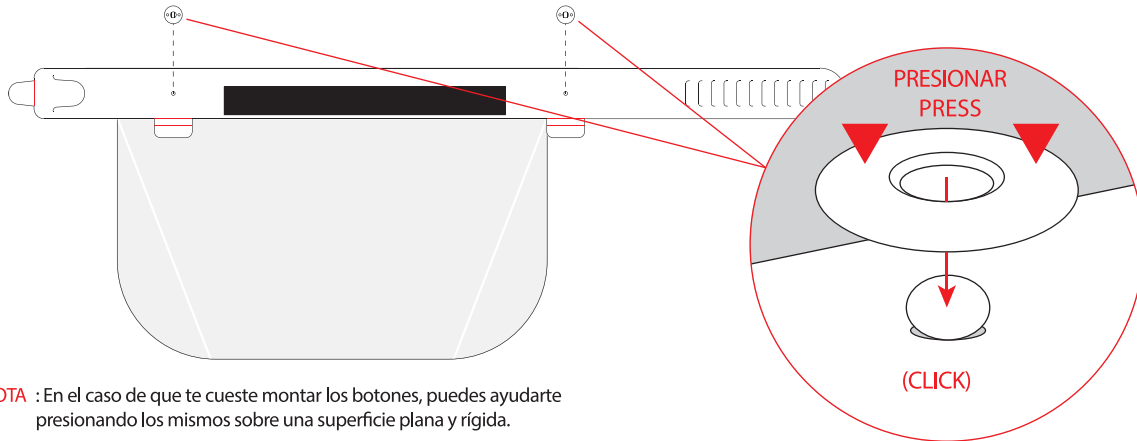
#### BUTTON CLOSURE

For your safety and securing of the face shield, place the two bottom parts of the buttons into the stems of the top buttons and press hard to fix them into place.

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
**NOTE** : In case you find it difficult to insert the buttons, you may find it easier by pressing them down on a hard flat surface, or by using a small hammer or pliers enabling you to press down harder.

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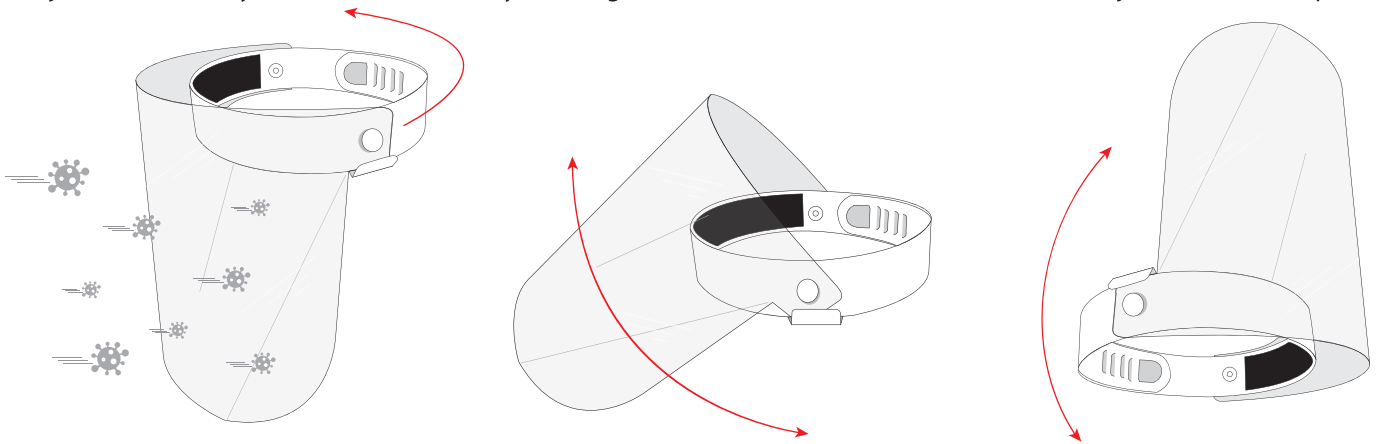
#### MASK HOLDER INCLUDED ON THE ITEM

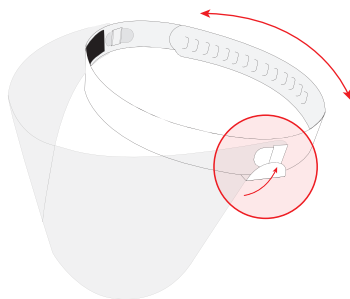Fold the two side tabs upwards, that you will find on the head strap and use them as a hook for the elastic of the mask, therefore protecting the ears.

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The face shield protects the wearer from splashes and particles, in particular around the sensitive areas such as the eyes, nose and mouth. In addition, it fits comfortably and provides a wide field of vision. It is not Personal Protective Equipment (PPE) nor is it classed as a comprehensive protection system. It is not covered by rule 2016/425. However, it can be used with other protective items, such as masks or hats, as an additional accessory. Using a face shield provides an additional measure of hygiene when social distancing or when using a mask.