S-BASIC

CABINA INSONORIZADA
AUDIOMETRIC TEST BOOTH
CABINE AUDIOMÉTRIQUE

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Revision: 501-800-MUM Rev 2.03

All rights reserved

This manual can be purchased through the After Sales Service.

 **SIBEL S.A.U.**

Rosellón 500 bajos, 08026 BARCELONA (Spain)

National Sales: Tel. +34 93 436 00 08 e-mail: comercial@sibelmed.comInternational Sales: Tel. +34 93 436 00 07 e-mail: export@sibelmed.comTechnical Service: Tel. +34 93 433 54 50 e-mail: sat@sibelmed.comFax: +34 93 436 16 11, Web: www.sibelmed.com**COPYRIGHT**

No part of this publication may be reproduced, transmitted, transcribed, stored in a back-up system or translated into any language or computer language in any form or by any means, electronic, mechanical, optical, chemical or manual without the express written consent from [SIBEL S.A.U.](#)

DISCLAIMER

SIBEL S.A.U. is responsible for the security, reliability and performance of this product only if:

- The place where the system is installed meets the requirements of electric installation IEC and other applicable regulations.
- Repairs, revisions or modifications, within and outside the warranty period are carried out by technical personnel of [SIBEL S.A.U.](#)
- The product is used by qualified staff in accordance with the recommendations of this User's Manual.

Thank you for choosing **SIBELMED S-BASIC**

The **SIBELMED S-BASIC** soundproof booths have been designed by the **R&D** department of **SIBEL S.A.U.**, using the latest materials in acoustic insulation.

This product is manufactured in accordance with the European Regulation of Medical Devices and with the best quality standards.

The CE mark consists of two stylized, bold, black letters 'C' and 'E' joined together.

**PRODUCT IN COMPLIANCE WITH MEDICAL
DEVICE DIRECTIVE 93/42/EEC. (Class I).**

Revised
Date: 2020-11
Technical Director

Approved
Date: 2020-11
Commercial Director

INDEX

1. SAFETY	4
1.1 INTENDED USE.....	4
1.2 INDICATIONS FOR USE.....	4
1.3 PRELIMINARY OBSERVATIONS	4
1.4. WARNINGS	4
1.5. DISPOSAL OF ELECTRICAL AND ELECTRONIC DEVICES BY DOMESTIC USERS IN THE EUROPEAN UNION	5
2. INSTRUCTIONS FOR USE AND INSTALLATION.....	5
2.1 PRINCIPLES OF OPERATION	5
2.2 MAIN FEATURES	5
2.3 PACKING LIST	6
2.4 BOOTH LAYOUT	7
2.5 INSTALLATION AND START-UP	8
2.5.1 INSTALLATION	8
2.5.2 ASSEMBLY GUIDE.....	9
2.5.3 START-UP.....	10
3. CLEANING AND MAINTENANCE.....	10
3.1 CLEANING.....	10
3.2 PREVENTIVE MAINTENANCE	10
3.2.1. SUPERVISION.....	10
3.2.2. INTERCONNECTION REVIEW.....	11
3.3. CORRECTIVE MAINTENANCE.....	11
4. TECHNICAL SPECIFICATIONS.....	11
5. APPLICABLE DIRECTIVES AND STANDARDS	13
6. SYMBOLS.....	14

1. SAFETY

The **SIBELMED S-BASIC** soundproof cabin has been designed for maximum safety. Read all instructions for use before using the cabin. Failure to do so may result in injury to the user or the patient and damage to the product and / or accessories.

You can obtain additional product training by contacting SIBEL, S.A.U. or your usual provider.

1.1 INTENDED USE

Soundproof individual cabin for audiometric testing.

1.2 INDICATIONS FOR USE

The **SIBELMED S-BASIC** soundproof cabin is an individual camera that attenuates sound waves between its exterior and interior. This characteristic of sound attenuation allows audiometry tests to be carried out under ambient noise conditions specified by the standards regarding how these types of measurements should be carried out. (For more information consult the ISO 8253-1 and ANSI S3.1 standards).

The cabin has a lockable access door, a window that allows mutual viewing between patient and operator, pleasant lighting and conditioned means to make the necessary connections, allowing the connection of an audiometer with all its accessories between the inside and outside of the cabin.

1.3 PRELIMINARY OBSERVATIONS

This cabin has been manufactured following strict quality controls. However, accidents can happen in the transport or storage of the cabins, so it is advisable to make an initial review of its condition before installing it, as well as the accessories that complement it.

1.4 WARNINGS

If you detect any deterioration in the packaging, immediately contact the transport agency and your distributor before proceeding to install it. It should not come off the packaging, bags, etc., until you fully verify the correct operation of the equipment.

The cabin must be used in places protected from the weather.

Smoking is not allowed in the cabin.

The cabin should not be used in places rich in oxygen.

The shelf is designed to support a weight of less than 30 kg (enough to support the audiometer), care must be taken not to catch your fingers when operating the shelf.

If you have any questions during the installation process or at any other time, contact [SIBEL S.A.U.](#)'s after-sales service. or with your dealer.

1.5. DISPOSAL OF ELECTRICAL AND ELECTRONIC DEVICES BY DOMESTIC USERS IN THE EUROPEAN UNION

Do not dispose of the electrical part of this product together with household waste.

Conversely, if you must dispose of this type of waste, it is the user's responsibility to disassemble the panel and the electrical part of the cabin and deliver it to a designated collection point for recycling electronic and electrical equipment. Recycling and separate collection of these wastes at the time of disposal will help preserve natural resources and ensure that recycling protects health and the environment.

You can obtain more information on the proper disposal of the product and its accessories by contacting the Customer Service of [SIBEL S.A.U.](#) or with your dealer.

2. INSTRUCTIONS FOR USE AND INSTALLATION

2.1 PRINCIPLES OF OPERATION

According to the audiometric standards ISO 8253-1 and ANSI S3.1, audiometric tests must be performed under conditions that ambient noise does not mask the results of these tests.

For this, the cabin is an acoustically attenuated enclosure in which the background noise is much lower than the noise from the outside, allowing the audiometric tests to be carried out correctly.

2.2 MAIN FEATURES

The S-Basic soundproof cabin is available in 3 models with the following features and options:

	S-BASIC	S-BASIC B	S-BASIC C
Code	07322	09603	09604
External measurements (cm)	96 x 96 x 208 cm	110 x 110 x 208 cm	120 x 120 x 208 cm
Folding table			
6 connection panel			
Interior lighting			
Power 220 V			
Power 110 V			

Included Optional

2.3 PACKING LIST

DESCRIPTION	Quantity
Base panel	1
Front panel	1
Door panel	1
Side panel	1
Rear panel	1
Top panel	1
Folding table	1
Hooks	2
Side panel fixing screws, m6x35	24
Washers m7	24
Top panel fixing screws	12
Corners with adhesive tape	4
Network cable *	1
Connection cables with jack 6.3	5
Connection cable with jack 3.5	1
Amp*	1
Phillips-flat screwdriver	1
Allen key #5	1
User 's manual	1

*According to 220V or 110V power option

2.4 BOOTH LAYOUT

Nº 1 - Front Panel

Nº 2 - Connections panel

Nº 3 - Network connection and light switch

Nº 4 - Window

Nº 5 - Corner protectors

Nº 6 - Top Panel

Nº 7 - Top panel screws

Nº 8 - Side Panel

Nº 9 - Rear Panel

Nº 10 - Booth base

Nº 11 - Door frame

Nº 12 - Door

Nº 13 - Door lock

Nº 14 - Window frame

Nº 15 - Network input

Nº 16 - Fold-down table

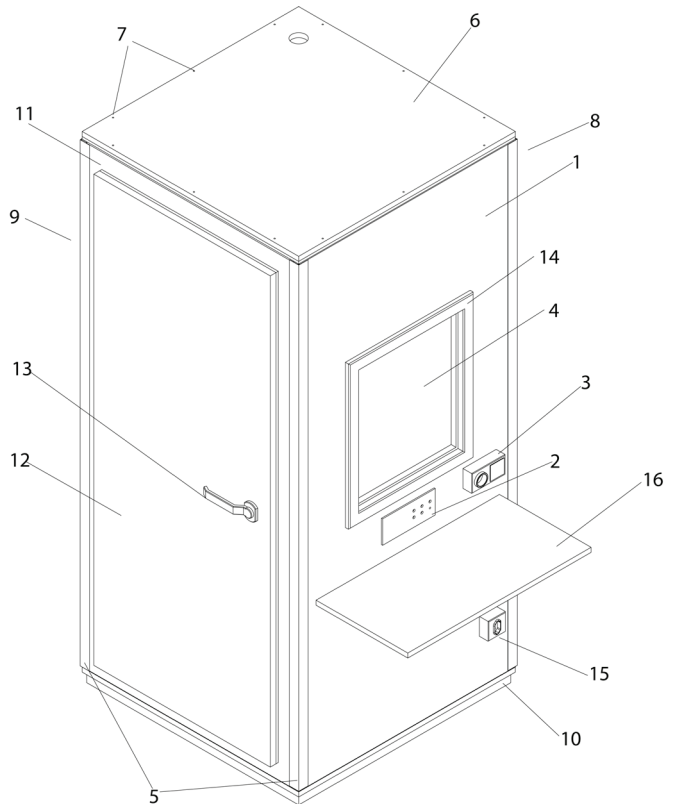
Figura 1.1. Overview

2.5 INSTALLATION AND START-UP

2.5.1 INSTALLATION

To continue with the assembly of the booth, follow the steps shown in the following chart:

Tools for assembly


NOTE:

If the door is not assembled for reasons of transportation, or you want to change the direction it opens in (Fig. 3), take the following steps:

- To change the opening direction, turn the panel 180°.
- To mount the door, place the hinges of the door facing the hinges of the **Panel** and insert the pins. Use a hammer to push the pins all the way through.
- Check the position of the closing wedge and reverse it if necessary.
- Make sure that the door is adjusted and closes properly.

2.5.2 ASSEMBLY GUIDE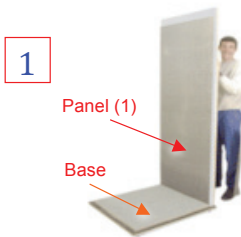

- 1
- A) Place the **Base** with its legs (or wheels) against the floor and as near as possible to its final position.
- B) Place the **Panel (1)** centered over the rubber of the **Base**, taking into account the final layout for the **Door Panel** and **Window Panel**.

2

Place the **Panel (2)** centered on the rubber of the **Base**. Make sure it is aligned and place the corresponding screws, but do not tighten.

3

Repeat the procedure with the **Door Panel** or **Window Panel** (depending on the final layout), and place the corresponding screws, but do not tighten.

4

After this, place the **Remaining panel**, and the corresponding screws, but do not tighten them.

5

Place the **Ceiling** on the booth (it will rest by its own weight, so it is not attached by screws). If the room is not high enough to allow this procedure, place the ceiling before placing the last panel by sliding it over the sides (See Figure 3).

The screws can now be tightened *****(side panels and ceiling) and the corner protectors placed over the corners.

2.5.3 START-UP

Take the steps below to start up the booth:

NOTE: It is understood that the user has an audiometer to connect to the booth.

- Connect the network cable to the base of the network input base of the cabin and then connect it to the network (No.15).
- Verify that the light switches on when the light switch (No. 3) is switched on. If the light does not switch on, check the previous step and verify that the power supply is working properly.
- Place the audiometer on the tray and connect the network cable from it to the auxiliary input (No.3). **IMPORTANT NOTE: Maximum load at 500 W.**
- Connect the audiometer to the connections panel with the connection cables and jacks and then connect the accessories (headphones, patient warning, etc.) to the connections panel on the inside of the booth and hang them from the supports that are placed inside the panel on each side.
- Switch on the main switch of the audiometer. Make sure that the audiometer works properly and as indicated in the user manual.

3. CLEANING AND MAINTENANCE

3.1 CLEANING

The exterior surface of the cabin can be cleaned with soap and water, subsequently drying the remaining moisture. Inside the cabin, the foam walls can be cleaned with a conventional vacuum cleaner; the plastic parts, the window and the washable floor can be washed with soap and water. Do not use abrasive substances or solvents that can attack the finish of the cabin.

3.2 PREVENTIVE MAINTENANCE

3.2.1. SUPERVISION

The user can periodically monitor the appearance of the cabin, the internal lighting, the door closing and the status of the grid connection, the auxiliary socket and the switch for the internal lighting. It will verify that no element presents breakage or external damage.

3.2.2. INTERCONNECTION REVIEW

Every six months it can be verified with the help of an audiometer that the wiring through the cabin is correct and that they do not produce noise that could reduce the reliability of the audiometric tests.

3.3. CORRECTIVE MAINTENANCE

In the event of detecting a breakdown in the equipment that prevents its normal use, contact [Sibel S.A.U.'s](#) after-sales service, specifying in as much detail as possible the type of anomaly that has occurred.

4. SPECIFICATIONS

	S-BASIC	S-BASIC B	S-BASIC C
External measures	96 x 96 x 208 cm	110 x 110 x 208 cm	120 x 120 x 208 cm
Internal measures	86 x 86 x 190 cm	101 x 101 x 193 cm	111 x 111 x 193 cm
Window	70 x 58 cm		
Door	80 x 188 cm		
Folding table	80 x 40 cm		
Weight	< 200 Kg	<220 Kg	<240 Kg
Internal supports	2		
Power supply	220 V 50/60 Hz. (Other optional voltages)		
Internal lighting	Circular LED lamp. Model TC-0159-BLA		
Environmental conditions Use: Storage and transportation:	5 °C a 40 °C, 5% a 85% RH without condensation -5 °C a 45 °C, 5% a 95% RH without condensation		


The characteristics of the acoustic insulation of the S-BASIC cabin (model only 96 x 96 x 208 cm) have been determined using the ISO 11957 standard, obtaining an attenuation curve for a frequency range between 100 and 10,000 Hz.

f (Hz)	100	125	160	200	250	315	400
D (dB)	15,1	20,9	16,3	17,9	21,3	27,2	21,5

f (Hz)	500	630	800	1.000	1.250	1.600	2.000
D (dB)	23,8	25,9	28,2	27,2	27,6	29,7	30,6

f (Hz)	2.500	3.150	4.000	5.000	6.300	8.000	10.000
D (Hz)	31,6	33,9	36,1	39,0	40,8	40,1	41,6

Test performed in Applus - L.G.A.I. - Certified Exp. No. 10/101534-1069

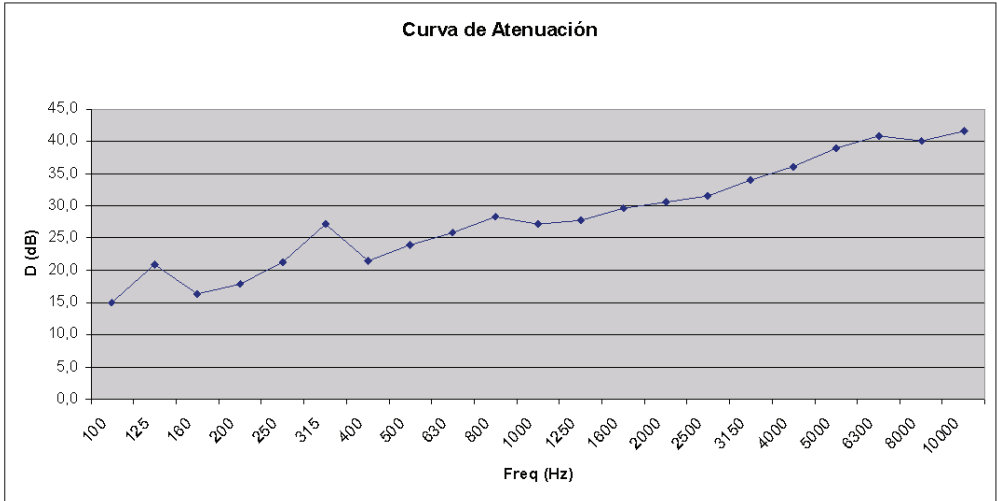Attenuation value at 8,000Hz = 40 dB. Global attenuation value Dw = 28 dB
 Calculated according to standard ISO 717-1, using the curve values up to 5,000 Hz.

5. APPLICABLE DIRECTIVES AND STANDARDS

- European Medical Device Directive 93/42/EEC
- RoHs Directive 2011/65/UE (RD 219/2013)
- Directive 2002/96/EC on Waste from electrical and electronic equipment (WEEE) (RD 208/2005)
- Audiometry: Allows compliance with standards ISO 8253-1 and ANSI S3.1 depending on environmental conditions.
- Quality: Sibel Quality System (EN ISO 9001:2015 e EN ISO 13485:2016+AC:2016)
- Safety: EN 60601-1:2006+AC:2010 + A1:2013, EN 60601-1-6:2010 + A1:2015; EN 62366:2008 + A1:2015
- Risk Management: EN ISO 14971:2012
- Information provided by the manufacturer: EN 1041:2008
- Symbols used: EN 15223-1:2016
- Acoustic isolation: EN ISO 11957:2009

6. SYMBOLS

MANUFACTURER (manufacture date,
name and address of the manufacturer)

CATALOGUE NUMBER

SERIAL NUMBER

BATCH CODE

CE MARKING

CAUTION, ADDITIONAL WARNINGS IN INSTRUCTIONS FOR
USE

FOLLOW INSTRUCTIONS FOR USE

WARNING, RISK IDENTIFIED

WASTE OF ELECTRICAL AND ELECTRONIC EQUIPMENT

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